

Boule Con-Diff

CONTROL

IVD



Open vial stability 14 days ¹

Control	Lot	
Tri-Level	---	---
Low	22312-41	2024-05-03
Normal	22312-42	2024-05-03
High	22312-43	2024-05-07

MEDONIC

CONTROL		L	
LOT		22312-41	
Medonic M-series/M32		Mean ²	± Range ³
RBC	10 ¹² /L	2.22	0.12
MCV	fL	63.9	5.0
RDW	%	14.6	4.0
Hct	%	14.2	2.5
Plt	10 ⁹ /L	83	15
MPV	fL	11.1	1.5
WBC	10 ⁹ /L	3.3	0.3
Hgb	g/dL	5.3	0.3
MCH	pg	23.9	1.5
MCHC	g/dL	37.3	3.0
Lymph#	10 ⁹ /L	1.5	0.4
Mid#	10 ⁹ /L	0.2	0.3
Gran#	10 ⁹ /L	1.6	0.8
Lymph	%	45.7	7.0
Mid	%	5.0	5.0
Gran	%	49.3	7.0

CONTROL		N	
LOT		22312-42	
		Mean ²	± Range ³
		4.20	0.18
		86.9	5.0
		13.1	4.0
		36.5	3.0
		234	30
		11.3	1.5
		8.4	0.6
		12.8	0.4
		30.5	1.5
		35.1	3.0
		4.0	0.6
		0.4	0.5
		4.0	1.0
		47.6	5.0
		5.0	5.0
		47.4	5.0

CONTROL		H	
LOT		22312-43	
		Mean ²	± Range ³
		5.13	0.22
		92.4	5.0
		13.5	4.0
		47.4	3.5
		519	60
		11.6	1.5
		20.9	1.8
		16.4	0.5
		32.0	1.5
		34.6	3.0
		12.9	1.5
		1.0	1.0
		7.0	2.5
		61.5	5.0
		5.0	5.0
		33.5	5.0

1

bg	Трайност на отворен флакон 14 дни (bg)
hr	Stabilnost otvorene bočice 14 dni (hr)
cs	Stabilita otevřené ampulky 14 dní (cs)
da	Åben rør stabilitet 14 dage (da)
et	Stabiilsus avatud viaalis 14 päevade (et)
fr	Stabilité en flacon ouvert 14 jour (fr)
de	Stabilität geöffneter Flaschen 14 tage (de)
el	Διάρθρωση δείγματος μετά την αποσφράγιση 14 ημέρα (el)
it	Stabilità della fiala aperta 14 giorni (it)
lv	Atvērt flakonu stabilitāti 14 dienas (lv)
lt	Stabilumas atidarius buteliuką 14 dienas (lt)
no	Åpen rør/glass stabilitet 14 dager (no)
pl	Trwałość po otwarciu fiołki 14 Liczba dni (pl)
pt	Estabilidade após abertura do frasco 14 dias (pt)
ro	Termenul de valabilitate al fiolei desfăcute 14 zile (ro)
ru	Стабильность вскрытого флакона 14 дней (ru)
sr	Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
sk	Stabilita otvorenej injekčnej liekovky 14 dní (sk)
sl	Stabilita otvorenej injekčnej liekovky 14 dni (sl)
es	Estabilidad de la cápsula abierta 14 días (es)
sv	Hållbarhet för öppnad flaska 14 dagar (sv)
tr	Açık şişe stabilitesi 14 gün (tr)

2

Sredno
Sredstvo
Průměr
Middelværdi
Keskmine
Moyenne
Mittelwert
Μέση
Media
Nozīmē
Vidurkis
Middelverdi
Średnia
Média
Medie
Среднее
Srednja vrednost
Priemer
Povprečno
Media
Medelvärde
Ortalama

3

Обхват
Raspon
Rozsah
Område
Vahemik
Intervalle
Bereich
Αναμενόμενο εύρος α
Intervallo
Diapazons
Intervalas
Område
Zakres
Intervalo
Interval
диапазон
Opseg
Rozsah
Območje
Intervalo
Intervall
Aralığı

For Medonic M-series / M32 with barcode reader

- M-Series:**
1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

- M32:**
1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.:	M-series	Instr.:	M-series	Instr.:	M-series
Lot:	22312-41	Lot:	22312-42	Lot:	22312-43
Prod.:	Con-Diff Low	Prod.:	Con-Diff Norm	Prod.:	Con-Diff High
ID:	2231241+	ID:	2231242+	ID:	2231243+
Exp.:	3-May-2024	Exp.:	3-May-2024	Exp.:	7-May-2024

1:	 B o u l e C B A V 2	 B o u l e C B A V 2	 B o u l e C B A V 2
2:	 2231241+	 2231242+	 2231243+
3:	 24050300001153030000	 24050300002153030000	 24050700003153030000
4:	 10004031800000000000	 10004031600000000000	 10004031100000000000
5:	 22212639500530314225	 42018869501280436530	 51322924501640547435
6:	 08315111152391537330	 23430113153051535130	 51960116153201534630
7:	 03303015040020301608	 08406040060040504010	 20918129150101007025
8:	 14640457700505049370	 13140476500505047450	 13540615500505033550
9:	 11116664630844583967	 11114618157493021393	 11111399598069494031