

### Instructions

For detailed instructions for use, please refer to the User Manual of the analyzer.



### Intended use

The Boule EasyCleaner 50 mL is intended for use with the Medonic M51 and Swelab Lumi hematology analyzers, for corrective and preventive cleaning of inlets and tubing in contact with blood (protein).



### Summary and principles

The Boule EasyCleaner is a cleaner containing hypochlorite in a stabilized solution with added surfactants. The hypochlorite and surfactants will dissolve protein and clots including those attached to surfaces facilitating the removal during rinsing. The cleaner has the ability to act as a disinfectant, significantly reducing the number of viable microorganisms. Using this cleaning solution at the stated regular intervals will ensure low background counts (PLT) and longer time between failures.

### Storage and stability

The Boule EasyCleaner are stable at least up until the expiration date as shown on labels when stored at 2–30 °C. Open container stability is 60 days if storage recommendations are followed.

### Notes

- The cleaner should be disposed of according to local regulations. When draining remaining solution through the sewage system, always use plenty of water. The plastic bottle can be recycled after rinsing.
- Solution passing through the analyzer during cleaning will be collected and disposed of as a part of the general waste from the analyzer.
- This product is intended for use as supplied. Adulteration by dilution or addition of any materials to the product invalidates any IVD use of the product.

### Precautions

- For in vitro diagnostic use only on analyzers as stated above.
- The Boule EasyCleaner is classified as hazardous to health and the environment according to CLP [Regulation (EC) no 1272/2008]. The required labeling is given below, detailed information is given in the Safety Data Sheet (SDS).

Hazard pictogram:



Signal word:

**Danger**

Hazard statements:

**H314** Causes severe skin burns and eye damage.

Precautionary statements:

**P280** Wear protective gloves/eye protection. **P303 + P361 + P353** IF ON SKIN (or hair): Remove immediately all contaminated clothing. Rinse skin with water/shower. **P305+P351+P338** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P310** Immediately call a POISON CENTER or doctor/physician. **P391** Collect spillage. **P405** Store locked up.

Substances in mixture:

Sodium hypochlorite, sodium hydroxide

### Ordering information and service

For international customers contact your local distributor for orders and support.

REF	Description	Packaging
1504513	Boule EasyCleaner	50 mL