



Supporting the safe and efficient use of blood donations

Efficient processing of donated blood and ensuring its safe use in transfusions require stringent hematology testing combined with rapid and accurate results. Thanks to its fast operation, micro-pipette adapter (MPA), highly accurate results, and built-in quality monitoring, Swelab™ Alfa Plus hematology analyzer helps blood banks in securing these vital objectives.

Rapid pre-donation testing of HGB/HCT, WBC, and PLT

Pre-donor testing helps confirm donor suitability. Key questions to answer include whether or not the individual is suitable for donation, what is the exact PLT count (needed for optimal processing), and does the donor have a possible infection. Swelab Alfa Plus quickly analyses and presents the HGB/HCT, PLT, and WBC parameters, supporting clinical decision-making.

Quality monitoring of key blood components

Accurate monitoring of platelet concentrate facilitates optimal pooling and the equivalent information for RBCs is needed to define the number of cells in each concentrate bag. Swelab Alfa Plus quickly generates accurate PLT-C and RBC-C results, all presented on the analyzer color touchscreen.



Swelab Alfa Plus hematology analyzer is well-suited for blood bank applications. The system provides both accurate CBC results and helps in the quality monitoring of PLT and RBC concentrates.

MPA function offers fast pre-donation testing

Swelab Alfa Plus features MPA capillary blood sampling. Simply make a finger-stick, draw blood into a special 20 µL micro-capillary tube, slide it into the adapter, and insert in the analyzer. About one minute later, you view all the key results on the touch-sensitive display. No preparation, no pre-dilution and no vacuum tubes are required. As well as making fast, pre-donation blood cell determinations, MPA also saves the vein for the donation.



Take a finger-stick blood sample.



The MPA functionality lets you analyze capillary samples directly and saves the vein for the donation.

High quality and reliability combined with convenient functions

- Exclusive blood bank application with special measurement of high PLT concentrations.
- PLT-C linearity up to $5000 \times 10^9/L$.
- RBC-C linearity up to $8 \times 10^{12}/L$.
- Results with histograms displayed in about a minute on a WVGA landscape view color screen.
- MPA functionality on all models except Basic.
- Motor-driven cap-piercing device as option.
- Auto Sampler model available.
- Barcode reader included for easy entry of patient ID.
- Memory for 50 000 samples.
- Coincidence correction for all parameters.
- Modern graphical interface and color touch screen.
- Shear valve aided sampling.
- Barcode entry of assay values for controls and calibrators.


Four Swelab Alfa Plus analyzer models span a wide range of easy-to-use functions. All deliver highly accurate and precise blood cell counts.



Swelab Alfa Plus Basic – even the basic model includes shear-valve technology.



Swelab Alfa Plus Standard – five-sample mixer is well-suited for the smaller laboratory.

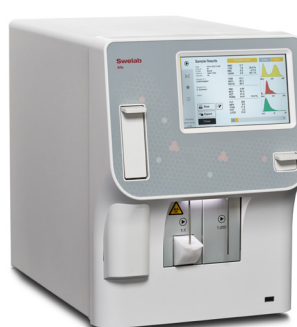

Swelab Alfa Plus Cap – closed-tube sampling minimizes risks from contaminated blood.



Swelab Alfa Plus Sampler – for up to 2×20 samples. Just load and walk away.

Contact us today for more details of the Swelab Alfa Plus hematology analyzer and its blood bank applications.

boule.com

Swelab is a trademark of Boule Medical AB.
© 2017-2022 Boule Diagnostics AB
Boule Diagnostics AB, Domnarvsgatan 4, SE-163 53 Spånga, Sweden
BPS27724-3 12/2022