

Boule Con-Diff

CONTROL

IVD



Open vial stability 14 days ¹



Control	Lot	
Tri-Level	32412-5K	2025-05-06
Low	32412-51	2025-05-06
Normal	32412-52	2025-05-06
High	32412-53	2025-05-09

MEDONIC

CONTROL	L
LOT	32412-51

Medonic M-series/M32		Mean ²	± Range ³
RBC	10 ¹² /L	2.25	0.12
MCV	fL	64.9	7.0
RDW	%	14.9	4.5
Hct	%	14.6	3.0
Plt	10 ⁹ /L	88	15
MPV	fL	11.2	1.5
WBC	10 ⁹ /L	3.5	0.3
Hgb	g/dL	5.5	0.3
MCH	pg	24.4	2.0
MCHC	g/dL	37.7	4.0
Lymph#	10 ⁹ /L	1.5	0.4
Mid#	10 ⁹ /L	0.2	0.3
Gran#	10 ⁹ /L	1.8	0.8
Lymph	%	43.8	7.0
Mid	%	6.0	5.0
Gran	%	50.2	7.0

CONTROL	N
LOT	32412-52

	Mean ²	± Range ³
	4.23	0.18
	81.6	7.0
	14.5	4.5
	34.5	3.5
	239	30
	11.5	1.5
	8.7	0.6
	12.0	0.4
	28.4	2.0
	34.8	4.0
	4.2	0.6
	0.6	0.5
	3.9	1.0
	47.4	5.0
	7.2	5.0
	45.4	5.0

CONTROL	H
LOT	32412-53

	Mean ²	± Range ³
	5.10	0.22
	93.6	7.0
	14.0	4.5
	47.7	4.0
	533	60
	11.9	1.5
	20.9	1.8
	16.3	0.5
	32.0	2.0
	34.2	4.0
	12.8	1.5
	1.2	1.0
	6.9	2.5
	61.0	5.0
	5.8	5.0
	33.2	5.0

- 1
- bg Трайност на отворен флакон 14 дни (bg)
 - hr Stabilnost otvorene bočice 14 dani (hr)
 - cs Stabilita otevrené ampulky 14 dny (cs)
 - da Åben r r stabilitet 14 dage (da)
 - et Stabiilsus avatud vialis 14 p evade (et)
 - fr Stabilit  en flacon ouvert 14 jour (fr)
 - de Stabilit t ge ffneter Flaschen 14 tage (de)
 - el Διατ ρηση δειγματος μετ  την αποσφραγιστ  14 ημερ  (el)
 - it Stabilit  della fiala aperta 14 giorni (it)
 - lv Atv rt flakonu stabilit ti 14 dienas (lv)
 - lt Stabilumas atidarius buteliuk  14 dienas (lt)
 - no  pen r r/glass stabilitet 14 dager (no)
 - pl Trwa o c po otwarciu fiolek 14 Liczba dni (pl)
 - pt Estabilidade ap s abertura do frasco 14 dias (pt)
 - ro Termenul de valabilitate al fiolei desfacute 14 zile (ro)
 - ru Стабильность вскрытого флакона 14 дней (ru)
 - sr Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
 - sk Stabilita otvorenej injekcnej liekovky 14 dni (sk)
 - sl Stabilita otvorenej injekcnej liekovky 14 dni (sl)
 - es Estabilidad de la c psula abierta 14 dias (es)
 - sv H llbarhet f r  ppnad flaska 14 dagar (sv)
 - tr A ık ŐiŐe stabilitesi 14 g n (tr)

- 2
- Sredno
 - Sredstvo
 - Pr m r
 - Middelv rdi
 - Keskmine
 - Moyenne
 - Mittelwert
 - M Ő 
 - Media
 - Noz m 
 - Vidurkis
 - Middelverdi
 - Őrednia
 - M dia
 - Medie
 - Srednee
 - Srednja vrednost
 - Priemer
 - Povpre no
 - Media
 - Medelv rde
 - Ortalama

- 3
- Ob vat
 - Raspon
 - Rozsah
 - Omr de
 - Vahemik
 - Intervalle
 - Bereich
 - Αναμεν μενο  ρος α
 - Intervallo
 - Diapazons
 - Intervalas
 - Omr de
 - Zakres
 - Intervalo
 - Interval
 - диапазон
 - Opseg
 - Rozsah
 - Obmo je
 - Intervalo
 - Intervall
 - Aral đı

25122-4



[http://www.medonic.se/
support/download](http://www.medonic.se/support/download)



For Medonic M-series / M32 with barcode reader

M-Series:

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

M32:

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: M-series
 Lot: 32412-51
 Prod.: Con-Diff Low
 ID: 3241251+
 Exp.: 6-May-2025

Instr.: M-series
 Lot: 32412-52
 Prod.: Con-Diff Norm
 ID: 3241252+
 Exp.: 6-May-2025

Instr.: M-series
 Lot: 32412-53
 Prod.: Con-Diff High
 ID: 3241253+
 Exp.: 9-May-2025



Boule Con-Diff

CONTROL

IVD



Open vial stability 14 days ¹



Control	Lot	
Tri-Level	32412-5K	2025-05-06
Low	32412-51	2025-05-06
Normal	32412-52	2025-05-06
High	32412-53	2025-05-09

SWELAB

		CONTROL	L
		LOT	32412-51
Swelab Alfa/Alfa Plus		Mean ²	± Range ³
RBC	10 ¹² /L	2.26	0.12
MCV	fL	66.4	7.0
RDW	%	14.2	4.5
Hct	%	15.0	3.0
Plt	10 ⁹ /L	89	15
MPV	fL	12.1	1.5
WBC	10 ⁹ /L	3.4	0.3
Hgb	g/dL	5.5	0.3
MCH	pg	24.3	2.0
MCHC	g/dL	36.7	4.0
Lymph#	10 ⁹ /L	1.7	0.4
Mid#	10 ⁹ /L	0.1	0.3
Gran#	10 ⁹ /L	1.6	0.8
Lymph	%	47.7	7.0
Mid	%	4.2	5.0
Gran	%	48.1	7.0

		CONTROL	N
		LOT	32412-52
		Mean ²	± Range ³
		4.20	0.18
		82.6	7.0
		13.3	4.5
		34.7	3.5
		241	30
		12.5	1.5
		8.6	0.6
		12.1	0.4
		28.8	2.0
		34.9	4.0
		4.4	0.6
		0.4	0.5
		3.8	1.0
		51.1	5.0
		4.2	5.0
		44.7	5.0

		CONTROL	H
		LOT	32412-53
		Mean ²	± Range ³
		5.08	0.22
		94.6	7.0
		13.1	4.5
		48.1	4.0
		544	60
		12.4	1.5
		20.7	1.8
		16.3	0.5
		32.1	2.0
		33.9	4.0
		13.2	1.5
		0.8	1.0
		6.7	2.5
		63.9	5.0
		3.7	5.0
		32.4	5.0

- 1
- bg Трайност на отворен флакон 14 дни (bg)
 - hr Stabilnost otvorene bočice 14 dani (hr)
 - cs Stabilita otevrené ampulky 14 dny (cs)
 - da Åben r r stabilitet 14 dage (da)
 - et Stabiilsus avatud vialis 14 p evade (et)
 - fr Stabilit  en flacon ouvert 14 jour (fr)
 - de Stabilit t ge ffneter Flaschen 14 tage (de)
 - el Διατήρηση δείγματος μετά την αποσφράγιση 14 ημ ρα (el)
 - it Stabilit  della fiala aperta 14 giorni (it)
 - lv Atv rt flakonu stabilit ti 14 dienas (lv)
 - lt Stabilumas atidarius buteliuk  14 dienas (lt)
 - no Åpen r r/glass stabilitet 14 dager (no)
 - pl Trwa o c po otwarciu fiolek 14 dni (pl)
 - pt Estabilidade ap s abertura do frasco 14 dias (pt)
 - ro Termenul de valabilitate al fiolei desfacute 14 zile (ro)
 - ru Стабильность вскрытого флакона 14 дней (ru)
 - sr Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
 - sk Stabilita otvorenej injekcnej liekovky 14 dn  (sk)
 - sl Stabilita otvorenej injekcnej liekovky 14 dni (sl)
 - es Estabilidad de la c psula abierta 14 dias (es)
 - sv H llbarhet f r  ppnad flaska 14 dagar (sv)
 - tr A ık ŐiŐe stabilitesi 14 g n (tr)

- 2
- Средно
 - Sredstvo
 - Pr m r
 - Middelv rdi
 - Keskm ne
 - Moyenne
 - Mittelwert
 - M s 
 - Media
 - Noz m 
 - Vidurkis
 - Middelverdi
 - Őrednia
 - M dia
 - M die
 - Среднее
 - Srednja vrednost
 - Priemer
 - Povpre no
 - Media
 - Medelv rde
 - Ortalama

- 3
- Обхват
 - Raspon
 - Rozsah
 - Omr de
 - Vahemik
 - Intervalle
 - Bereich
 - Αναμενόμενο ε ρος α
 - Intervallo
 - Diapazons
 - Intervalas
 - Omr de
 - Zakres
 - Intervalo
 - Interval
 - диапазон
 - Orseg
 - Rozsah
 - Obmo je
 - Intervalo
 - Intervall
 - Aral đı

25122-4



www.swelab.com/
support/download



For Swelab Alfa / Alfa Plus with barcode reader

Alfa:

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

Alfa Plus:

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: Swelab Alfa
Lot: 32412-51
Prod.: Con-Diff Low
ID: 3241251+
Exp.: 6-May-2025

Instr.: Swelab Alfa
Lot: 32412-52
Prod.: Con-Diff Norm
ID: 3241252+
Exp.: 6-May-2025

Instr.: Swelab Alfa
Lot: 32412-53
Prod.: Con-Diff High
ID: 3241253+
Exp.: 9-May-2025

