

# Boule Con-Diff

CONTROL

IVD



Open vial stability 14 days <sup>1</sup>



Control	Lot	
Tri-Level	32503-5K	2025-07-31
Low	32503-51	2025-07-31
Normal	32503-52	2025-08-01
High	32503-53	2025-08-08

## MEDONIC

CONTROL	L
LOT	32503-51

Medonic M-series/M32		Mean <sup>2</sup>	± Range <sup>3</sup>
RBC	10 <sup>12</sup> /L	2.26	0.12
MCV	fL	63.7	7.0
RDW	%	14.8	4.5
Hct	%	14.4	3.0
Plt	10 <sup>9</sup> /L	85	15
MPV	fL	11.8	1.5
WBC	10 <sup>9</sup> /L	3.3	0.3
Hgb	g/dL	5.4	0.3
MCH	pg	23.9	2.0
MCHC	g/dL	37.5	4.0
Lymph#	10 <sup>9</sup> /L	1.4	0.4
Mid#	10 <sup>9</sup> /L	0.2	0.3
Gran#	10 <sup>9</sup> /L	1.7	0.8
Lymph	%	42.5	7.0
Mid	%	6.5	5.0
Gran	%	51.0	7.0

CONTROL	N
LOT	32503-52

		Mean <sup>2</sup>	± Range <sup>3</sup>
		4.16	0.18
		84.9	7.0
		12.1	4.5
		35.3	3.5
		241	30
		11.6	1.5
		8.4	0.6
		12.4	0.4
		29.8	2.0
		35.1	4.0
		3.9	0.6
		0.6	0.5
		3.9	1.0
		47.1	5.0
		6.8	5.0
		46.1	5.0

CONTROL	H
LOT	32503-53

		Mean <sup>2</sup>	± Range <sup>3</sup>
		5.07	0.22
		93.5	7.0
		13.5	4.5
		47.4	4.0
		533	60
		12.0	1.5
		20.2	1.8
		16.3	0.5
		32.1	2.0
		34.4	4.0
		12.4	1.5
		1.2	1.0
		6.6	2.5
		61.5	5.0
		5.9	5.0
		32.6	5.0

- 1
- bg Трайност на отворен флакон 14 дни (bg)
  - hr Stabilnost otvorene bočice 14 dani (hr)
  - cs Stabilita otevrené ampulky 14 dny (cs)
  - da Åben r r stabilitet 14 dage (da)
  - et Stabiilsus avatud vialis 14 p evade (et)
  - fr Stabilit  en flacon ouvert 14 jour (fr)
  - de Stabilit t ge ffneter Flaschen 14 tage (de)
  - el Διατ ρηση δειγματος μετ  την αποσφραγιστ  14 ημερ  (el)
  - it Stabilit  della fiala aperta 14 giorni (it)
  - lv Atv rt flakonu stabilit ti 14 dienas (lv)
  - lt Stabilumas atidarius buteliuk  14 dienas (lt)
  - no  pen r r/glass stabilitet 14 dager (no)
  - pl Trwa o c po otwarciu fiolek 14 Liczba dni (pl)
  - pt Estabilidade ap s abertura do frasco 14 dias (pt)
  - ro Termenul de valabilitate al fiolei desfacute 14 zile (ro)
  - ru Стабильность вскрытого флакона 14 дней (ru)
  - sr Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
  - sk Stabilita otvorenej injekcnej liekovky 14 dni (sk)
  - sl Stabilita otvorenej injekcnej liekovky 14 dni (sl)
  - es Estabilidad de la c psula abierta 14 dias (es)
  - sv H llbarhet f r  ppnad flaska 14 dagar (sv)
  - tr A ık ŐiŐe stabilitesi 14 g n (tr)

- 2
- Sredno
  - Sredstvo
  - Pr m r
  - Middelv rdi
  - Keskmine
  - Moyenne
  - Mittelwert
  - M Őh
  - Media
  - Noz m 
  - Vidurkis
  - Middelverdi
  - Őrednia
  - M dia
  - Medie
  - Srednee
  - Srednja vrednost
  - Priemer
  - Povpre no
  - Med a
  - Medelv rde
  - Ortalama
- 3
- Ob vat
  - Raspon
  - Rozsah
  - Omr de
  - Vahemik
  - Intervalle
  - Bereich
  - Αναμεν μενο  ρος α
  - Intervallo
  - Diapazons
  - Intervalas
  - Omr de
  - Zakres
  - Intervalo
  - Interval
  - диапазон
  - Opseg
  - Rozsah
  - Obmo je
  - Intervalo
  - Intervall
  - Aralıđı

25122-4



[http://www.medonic.se/  
support/download](http://www.medonic.se/support/download)

**Boule**  
Boule Medical AB  
Domnarvsgatan 4  
SE-163 53 Sp nga,  
Sweden

# For Medonic M-series / M32 with barcode reader

**M-Series:**

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

**M32:**

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: M-series  
 Lot: 32503-51  
 Prod.: Con-Diff Low  
 ID: 3250351+  
 Exp.: 31-Jul-2025

Instr.: M-series  
 Lot: 32503-52  
 Prod.: Con-Diff Norm  
 ID: 3250352+  
 Exp.: 1-Aug-2025

Instr.: M-series  
 Lot: 32503-53  
 Prod.: Con-Diff High  
 ID: 3250353+  
 Exp.: 8-Aug-2025



# Boule Con-Diff

CONTROL

IVD



Open vial stability 14 days <sup>1</sup>



Control	Lot	
Tri-Level	32503-5K	2025-07-31
Low	32503-51	2025-07-31
Normal	32503-52	2025-08-01
High	32503-53	2025-08-08

## SWELAB

		CONTROL	L
		LOT	32503-51
Swelab Alfa/Alfa Plus		Mean <sup>2</sup>	± Range <sup>3</sup>
RBC	10 <sup>12</sup> /L	2.28	0.12
MCV	fL	64.5	7.0
RDW	%	14.3	4.5
Hct	%	14.7	3.0
Plt	10 <sup>9</sup> /L	87	15
MPV	fL	12.5	1.5
WBC	10 <sup>9</sup> /L	3.4	0.3
Hgb	g/dL	5.5	0.3
MCH	pg	24.1	2.0
MCHC	g/dL	37.4	4.0
Lymph#	10 <sup>9</sup> /L	1.6	0.4
Mid#	10 <sup>9</sup> /L	0.1	0.3
Gran#	10 <sup>9</sup> /L	1.7	0.8
Lymph	%	46.9	7.0
Mid	%	4.3	5.0
Gran	%	48.8	7.0

		CONTROL	N
		LOT	32503-52
		Mean <sup>2</sup>	± Range <sup>3</sup>
		4.16	0.18
		85.7	7.0
		11.3	4.5
		35.7	3.5
		252	30
		12.3	1.5
		8.4	0.6
		12.5	0.4
		30.0	2.0
		35.0	4.0
		4.2	0.6
		0.3	0.5
		3.9	1.0
		50.4	5.0
		3.7	5.0
		45.9	5.0

		CONTROL	H
		LOT	32503-53
		Mean <sup>2</sup>	± Range <sup>3</sup>
		5.09	0.22
		94.5	7.0
		12.7	4.5
		48.1	4.0
		549	60
		12.7	1.5
		20.1	1.8
		16.3	0.5
		32.0	2.0
		33.9	4.0
		12.9	1.5
		0.7	1.0
		6.5	2.5
		64.3	5.0
		3.4	5.0
		32.3	5.0

- 1
- bg Трайност на отворен флакон 14 дни (bg)
  - hr Stabilnost otvorene bočice 14 dani (hr)
  - cs Stabilita otevřené ampulky 14 dny (cs)
  - da Åben rør stabilitet 14 dage (da)
  - et Stabiilsus avatud vialis 14 päevade (et)
  - fr Stabilité en flacon ouvert 14 jour (fr)
  - de Stabilität geöffneter Flaschen 14 tage (de)
  - el Διατήρηση δείγματος μετά την αποσφράγιση 14 ημέρα (el)
  - it Stabilità della fiala aperta 14 giorni (it)
  - lv Atvērt flakonu stabilitāti 14 dienas (lv)
  - lt Stabilumas atidarius buteliuką 14 dienas (lt)
  - no Åpen rør/glass stabilitet 14 dager (no)
  - pl Trwałość po otwarciu fiołki 14 dni (pl)
  - pt Estabilidade após abertura do frasco 14 dias (pt)
  - ro Termenul de valabilitate al fiolei desfăcute 14 zile (ro)
  - ru Стабильность вскрытого флакона 14 дней (ru)
  - sr Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
  - sk Stabilita otvorenej injekčnej liekovky 14 dní (sk)
  - sl Stabilita otvorenej injekčnej liekovky 14 dni (sl)
  - es Estabilidad de la cápsula abierta 14 días (es)
  - sv Hållbarhet för öppnad flaska 14 dagar (sv)
  - tr Açık şişe stabilitesi 14 gün (tr)

- 2
- Средно
  - Sredstvo
  - Průměr
  - Middelværdi
  - Keskmîne
  - Moyenne
  - Mittelwert
  - Μέση
  - Media
  - Nozimë
  - Vidurkis
  - Middelverdi
  - Średnia
  - Média
  - Medie
  - Среднее
  - Srednja vrednost
  - Priemer
  - Povprečno
  - Media
  - Medelvärde
  - Ortalama

- 3
- Обхват
  - Raspon
  - Rozsah
  - Område
  - Vahemik
  - Intervalle
  - Bereich
  - Αναμενόμενο εύρος α
  - Intervallo
  - Diapazons
  - Intervalas
  - Område
  - Zakres
  - Intervalo
  - Interval
  - диапазон
  - Orseg
  - Rozsah
  - Območje
  - Intervalo
  - Intervall
  - Aralığı

25122-4



[www.swelab.com/](http://www.swelab.com/)  
support/download



## For Swelab Alfa / Alfa Plus with barcode reader

### Alfa:

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

### Alfa Plus:

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: Swelab Alfa  
Lot: 32503-51  
Prod.: Con-Diff Low  
ID: 3250351+  
Exp.: 31-Jul-2025

Instr.: Swelab Alfa  
Lot: 32503-52  
Prod.: Con-Diff Norm  
ID: 3250352+  
Exp.: 1-Aug-2025

Instr.: Swelab Alfa  
Lot: 32503-53  
Prod.: Con-Diff High  
ID: 3250353+  
Exp.: 8-Aug-2025

