

CDS Hematology Calibrator

KIT LOT:

32503-34

EXP:

26-Jun-25

Open Vial Stability:

5 days

Analyzer	WBC x10 ⁹ /L	RBC x10 ¹² /L	Hgb g/dL	Hct %	MCV fL	PLT x10 ⁹ /L	MPV fL
Beckman Coulter Large Systems S Plus Series, JT Series, STKR, STKS™, MAXM™	8.3	4.26	12.3	35.0	82.2	240	11.5
Beckman Coulter Small Systems MD, MD II Ac-T™, Ac-T diff™ Series Onyx and T Series	8.2 8.2	4.23 4.23	12.3 12.3	35.2 35.2	83.1 83.1	234 234	11.3 11.3
Abbott Cell-Dyn® 1400, 1500, 1600, 1700 Cell-Dyn® 1800	8.4 8.0	4.34 4.33	12.4 12.5	35.2 35.5	81 82	253 239	na na
Sysmex KX Series	8.0	4.24	12.3	32.4	76.5	252	9.6
Mindray 3000 Series	8.1	4.41	12.3	37.6	85.2	276	10.4
ABX Micros 45/60	8.1	4.23	12.4	34.9	82.5	266	10.6
Orphee Mythic 18	8.2	4.21	12.2	36.4	86.5	245	11.4
Range Limits:	± 0.3	± 0.10	± 0.2	± 1.0	± 2.0	± 15	± 1.5

Range Limits are provided for evaluating analyzer data, and for determination of analyzer reproducibility and operator skill.

All trademarked analyzer names belong to the respective trademark holders.



Manufactured by:
Clinical Diagnostic Solutions, Inc.
1800 N. W. 65th Avenue
Plantation, FL 33313 U.S.A.

