

Boule Con-Diff

CONTROL

IVD



Open vial stability 14 days ¹

Control	Lot	
Tri-Level	22506-1K	2025-10-31
Low	22506-11	2025-10-31
Normal	22506-12	2025-10-31
High	22506-13	2025-11-04

MEDONIC

CONTROL		L	
LOT		22506-11	
Medonic M-series/M32		Mean ²	± Range ³
RBC	10 ¹² /L	2.24	0.12
MCV	fL	66.8	5.0
RDW	%	14.9	4.0
Hct	%	15.0	2.5
Plt	10 ⁹ /L	83	15
MPV	fL	10.7	1.5
WBC	10 ⁹ /L	3.4	0.3
Hgb	g/dL	5.5	0.3
MCH	pg	24.6	1.5
MCHC	g/dL	36.7	3.0
Lymph#	10 ⁹ /L	1.5	0.4
Mid#	10 ⁹ /L	0.2	0.3
Gran#	10 ⁹ /L	1.7	0.8
Lymph	%	44.2	7.0
Mid	%	5.7	5.0
Gran	%	50.1	7.0

CONTROL		N	
LOT		22506-12	
		Mean ²	± Range ³
		4.15	0.18
		79.8	5.0
		14.6	4.0
		33.1	3.0
		238	30
		11.2	1.5
		8.2	0.6
		11.7	0.4
		28.2	1.5
		35.3	3.0
		3.9	0.6
		0.5	0.5
		3.8	1.0
		46.7	5.0
		6.4	5.0
		46.9	5.0

CONTROL		H	
LOT		22506-13	
		Mean ²	± Range ³
		5.06	0.22
		92.7	5.0
		13.8	4.0
		46.9	3.5
		537	60
		11.4	1.5
		20.2	1.8
		16.2	0.5
		32.0	1.5
		34.5	3.0
		12.4	1.5
		1.2	1.0
		6.6	2.5
		61.5	5.0
		5.7	5.0
		32.8	5.0

- 1
- bg Трайност на отворен флакон 14 дни (bg)
 - hr Stabilität otvorene bočice 14 dni (hr)
 - cs Stabilita otevřených ampulky 14 dní (cs)
 - da Åben rør stabilitet 14 dage (da)
 - et Stabiilsus avatud vialis 14 päeva (et)
 - fr Stabilité en flacon ouvert 14 jour (fr)
 - de Stabilität geöffneten Flaschen 14 tage (de)
 - el Διατήρηση δείγματος μετά την αποσφράγιση 14 ημέρα (el)
 - it Stabilità della fiala aperta 14 giorni (it)
 - lv Atvērt flakonu stabilitāti 14 dienas (lv)
 - lt Stabilitumas atidarius buteliuką 14 dienas (lt)
 - no Åpen rør/glass stabilitet 14 dager (no)
 - pl Trwałość po otwarciu fiolek 14 dni (pl)
 - pt Estabilidade após abertura do frasco 14 dias (pt)
 - ro Termenul de valabilitate al fiolei deschise 14 zile (ro)
 - ru Стабильность вскрытого флакона 14 дней (ru)
 - sr Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
 - sk Stabilita otvorenej injekčnej liekovky 14 dní (sk)
 - sl Stabilita otvorene injekčneje liekovky 14 dni (sl)
 - es Estabilidad de la cápsula abierta 14 días (es)
 - sv Hållbarhet för öppnad flaska 14 dagar (sv)
 - tr Açık şişe stabilitesi 14 gün (tr)

- 2
- Sredno
 - Sredstvo
 - Průměr
 - Middelverdi
 - Keskmine
 - Moyenne
 - Mittelwert
 - Mέση
 - Media
 - Nozīmē
 - Vidurkis
 - Middelverdi
 - Średnia
 - Média
 - Medie
 - Среднее
 - Srednja vrednost
 - Priemer
 - Povprečno
 - Media
 - Medelvärde
 - Ortalama

- 3
- Обхват
 - Raspon
 - Rozsah
 - Område
 - Vahemik
 - Intervalle
 - Bereich
 - Αναμενόμενο εύρος α
 - Intervallo
 - Diapazons
 - Intervalas
 - Område
 - Zakres
 - Intervalo
 - Interval
 - диапазон
 - Opseg
 - Rozsah
 - Območje
 - Intervalo
 - Intervall
 - Aralığı

For Medonic M-series / M32 with barcode reader

M-Series:

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

M32:

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: M-series

Lot: 22506-11

Prod.: Con-Diff Low

ID: 2250611+

Exp.: 31-Oct-2025

Instr.: M-series

Lot: 22506-12

Prod.: Con-Diff Norm

ID: 2250612+

Exp.: 31-Oct-2025

Instr.: M-series

Lot: 22506-13

Prod.: Con-Diff High

ID: 2250613+

Exp.: 4-Nov-2025

1:



Boule CBA V2



Boule CBA V2



Boule CBA V2

2:



2250611+



2250612+



2250613+

3:



25103100001153030000



25103100002153030000



25110400003153030000

4:



10004030100000000000



10004029800000000000



10004029800000000000

5:



22412668500550315025



41518798501170433130



50622927501620546935

6:



08315107152461536730



23830112152821535330



53760114153201534530

7:



03403015040020301708



08206039060050503810



20218124150121006625

8:



14940442700575050170



14640467500645046950



13840615500575032850

9:



11118024971999675142



11119944840851537992



11110344069533685908

Boule Con-Diff

CONTROL

IVD



Open vial stability 14 days ¹

Control	Lot	
Tri-Level	22506-1K	2025-10-31
Low	22506-11	2025-10-31
Normal	22506-12	2025-10-31
High	22506-13	2025-11-04

SWELAB

CONTROL		L	
LOT		22506-11	
		Mean ²	± Range ³
RBC	10 ¹² /L	2.25	0.12
MCV	fL	68.6	5.0
RDW	%	14.0	4.0
Hct	%	15.4	2.5
Plt	10 ⁹ /L	87	15
MPV	fL	11.3	1.5
WBC	10 ⁹ /L	3.5	0.3
Hgb	g/dL	5.6	0.3
MCH	pg	24.9	1.5
MCHC	g/dL	36.4	3.0
Lymph#	10 ⁹ /L	1.7	0.4
Mid#	10 ⁹ /L	0.1	0.3
Gran#	10 ⁹ /L	1.7	0.8
Lymph	%	47.9	7.0
Mid	%	3.5	5.0
Gran	%	48.6	7.0

CONTROL		N	
LOT		22506-12	
		Mean ²	± Range ³
		4.17	0.18
		81.1	5.0
		13.1	4.0
		33.8	3.0
		244	30
		11.7	1.5
		8.3	0.6
		11.7	0.4
		28.1	1.5
		34.6	3.0
		4.2	0.6
		0.3	0.5
		3.8	1.0
		50.3	5.0
		3.8	5.0
		45.9	5.0

CONTROL		H	
LOT		22506-13	
		Mean ²	± Range ³
		5.05	0.22
		94.1	5.0
		12.5	4.0
		47.5	3.5
		549	60
		11.8	1.5
		20.1	1.8
		16.0	0.5
		31.7	1.5
		33.7	3.0
		12.9	1.5
		0.7	1.0
		6.5	2.5
		64.4	5.0
		3.5	5.0
		32.1	5.0

1
bg Трайност на отворен флакон 14 дни (bg)
hr Stabilität otvorene bočice 14 dni (hr)
cs Stabilita otevřených ampulky 14 dní (cs)
da Åben rør stabilitet 14 dage (da)
et Stabiilsus avatud viaalis 14 päevade (et)
fr Stabilité en flacon ouvert 14 jour (fr)
de Stabilität geöffneten Flaschen 14 tage (de)
el Διατήρηση δείγματος μετά την αποσφράγιση 14 ημέρα (el)
it Stabilità della fiala aperta 14 giorni (it)
lv Atvērt flakonu stabilitāti 14 dienas (lv)
lt Stabilumas atidarius buteliuką 14 dienas (lt)
no Åpen rør/glass stabilitet 14 dager (no)
pl Trwałość po otwarciu fiolek 14 dni (pl)
pt Estabilidade após abertura do frasco 14 dias (pt)
ro Termenul de valabilitate al fiolei desfăcute 14 zile (ro)
ru Стабильность вскрытого флакона 14 дней (ru)
sr Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
sk Stabilita otvorenej injekčnej liekovky 14 dní (sk)
sl Stabilita otvorene injekčneje liekovky 14 dni (sl)
es Estabilidad de la cápsula abierta 14 días (es)
sv Hållbarhet för öppnad flaska 14 dagar (sv)
tr Açık şişe stabilitesi 14 gün (tr)

2
Средно
Sredstvo
Průměr
Middelverdi
Keskmine
Moyenne
Mittelwert
Μέση
Media
Nozīmē
Vidurkis
Middelverdi
Średnia
Média
Medie
Среднее
Srednja vrednost
Priemer
Povprečno
Media
Medelvärde
Ortalama

3
Обхват
Raspon
Rozsah
Område
Vahemik
Intervalle
Bereich
Αναμενόμενο εύρος α
Intervallo
Diapazons
Intervalas
Område
Zakres
Intervalo
Interval
диапазон
Opseg
Rozsah
Območje
Intervalo
Intervall
Arahğı

For Swelab Alfa / Alfa Plus with barcode reader

Alfa:

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

Alfa Plus:

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: Swelab Alfa

Lot: 22506-11

Prod.: Con-Diff Low

ID: 2250611+

Exp.: 31-Oct-2025

Instr.: Swelab Alfa

Lot: 22506-12

Prod.: Con-Diff Norm

ID: 2250612+

Exp.: 31-Oct-2025

Instr.: Swelab Alfa

Lot: 22506-13

Prod.: Con-Diff High

ID: 2250613+

Exp.: 4-Nov-2025

1:



B o u l e C B A V 3



B o u l e C B A V 3



B o u l e C B A V 3

2:



2250611+



2250612+



2250613+

3:



25103100001153030000



25103100002153030000



25110400003153030000

4:



20014018000000000000



20014018000000000000



20014018000000000000

5:



22512686500560315425



41718811501170433830



50522941501600547535

6:



08715113152491536430



24430117152811534630



54960118153171533730

7:



03503017040010301708



08306042060030503810



20118129150071006525

8:



14040479700355048670



13140503500385045950



12540644500355032150

9:



11117592227329171801



11117447376700096941



11110386843799179309