

Boule Con-Diff

CONTROL

IVD



Open vial stability 14 days ¹



Control	Lot	
Tri-Level	32506-5K	2025-11-01
Low	32506-51	2025-11-01
Normal	32506-52	2025-11-05
High	32506-53	2025-11-07

MEDONIC

CONTROL		L	
LOT		32506-51	
Medonic M-series/M32		Mean ²	± Range ³
RBC	10 ¹² /L	2.20	0.12
MCV	fL	64.9	7.0
RDW	%	14.8	4.5
Hct	%	14.3	3.0
Plt	10 ⁹ /L	82	15
MPV	fL	10.6	1.5
WBC	10 ⁹ /L	3.4	0.3
Hgb	g/dL	5.4	0.3
MCH	pg	24.5	2.0
MCHC	g/dL	37.8	4.0
Lymph#	10 ⁹ /L	1.5	0.4
Mid#	10 ⁹ /L	0.2	0.3
Gran#	10 ⁹ /L	1.7	0.8
Lymph	%	44.2	7.0
Mid	%	5.8	5.0
Gran	%	50.0	7.0

CONTROL		N	
LOT		32506-52	
		Mean ²	± Range ³
		4.15	0.18
		79.4	7.0
		14.2	4.5
		33.0	3.5
		232	30
		10.6	1.5
		8.3	0.6
		11.6	0.4
		28.0	2.0
		35.2	4.0
		3.9	0.6
		0.5	0.5
		3.9	1.0
		46.2	5.0
		6.6	5.0
		47.2	5.0

CONTROL		H	
LOT		32506-53	
		Mean ²	± Range ³
		5.09	0.22
		90.1	7.0
		13.6	4.5
		45.9	4.0
		533	60
		11.1	1.5
		20.5	1.8
		16.0	0.5
		31.4	2.0
		34.9	4.0
		12.6	1.5
		1.2	1.0
		6.7	2.5
		61.4	5.0
		5.9	5.0
		32.7	5.0

1
bg Трайност на отворен флакон 14 дни (bg)
hr Stabilitnost otvorene bočice 14 dni (hr)
cs Stabilita otevřené ampulky 14 dní (cs)
da Åben rør stabilitet 14 dage (da)
et Stabiilsus avatud vialis 14 päevade (et)
fr Stabilité en flacon ouvert 14 jour (fr)
de Stabilität geöffneter Flaschen 14 tage (de)
el Διατήρηση δείγματος μετά την αποσφράγιση 14 ημέρα (el)
it Stabilità della fiala aperta 14 giorni (it)
lv Atvērt flakonu stabilitāti 14 dienas (lv)
lt Stabilumas atidarius buteliuką 14 dienas (lt)
no Åpen rør/glass stabilitet 14 dager (no)
pl Trwałość po otwarciu fiołki 14 Liczba dni (pl)
pt Estabilidade após abertura do frasco 14 dias (pt)
ro Termenul de valabilitate al fiolei desfăcute 14 zile (ro)
ru Стабильность вскрытого флакона 14 дней (ru)
sr Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
sk Stabilita otvorenej injekčnej liekovky 14 dní (sk)
sl Stabilita otvorenej injekčnej liekovky 14 dni (sl)
es Estabilidad de la cápsula abierta 14 días (es)
sv Hållbarhet för öppnad flaska 14 dagar (sv)
tr Açık şişe stabilitesi 14 gün (tr)

2
Средно
Sredstvo
Průměr
Middelverdi
Keskmine
Moyenne
Mittelwert
Μέση
Media
Nozīmē
Vidurkis
Middelverdi
Średnia
Média
Medie
Среднее
Srednja vrednost
Priemer
Povprečno
Media
Medelvärde
Ortalama

3
Обхват
Raspon
Rozsah
Område
Vahemik
Intervalle
Bereich
Αναμενόμενο εύρος α
Intervallo
Diapazons
Intervalas
Område
Zakres
Intervalo
Interval
диапазон
Opseg
Rozsah
Območje
Intervalo
Intervall
Aralığı

25122-4



[http://www.medonic.se/
support/download](http://www.medonic.se/support/download)

Boule
Boule Medical AB
Domnarvsgatan 4
SE-163 53 Spånga,
Sweden

For Medonic M-series / M32 with barcode reader

M-Series:

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

M32:

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: M-series
Lot: 32506-51
Prod.: Con-Diff Low
ID: 3250651+
Exp.: 1-Nov-2025

Instr.: M-series
Lot: 32506-52
Prod.: Con-Diff Norm
ID: 3250652+
Exp.: 5-Nov-2025

Instr.: M-series
Lot: 32506-53
Prod.: Con-Diff High
ID: 3250653+
Exp.: 7-Nov-2025



3.

Boule Con-Diff

CONTROL

IVD



Open vial stability 14 days ¹



Control	Lot	
Tri-Level	32506-5K	2025-11-01
Low	32506-51	2025-11-01
Normal	32506-52	2025-11-05
High	32506-53	2025-11-07

SWELAB

CONTROL		L	
LOT		32506-51	
Swelab Alfa/Alfa Plus		Mean ²	± Range ³
RBC	10 ¹² /L	2.21	0.12
MCV	fL	66.6	7.0
RDW	%	14.0	4.5
Hct	%	14.7	3.0
Plt	10 ⁹ /L	81	15
MPV	fL	11.1	1.5
WBC	10 ⁹ /L	3.4	0.3
Hgb	g/dL	5.4	0.3
MCH	pg	24.4	2.0
MCHC	g/dL	36.7	4.0
Lymph#	10 ⁹ /L	1.6	0.4
Mid#	10 ⁹ /L	0.1	0.3
Gran#	10 ⁹ /L	1.7	0.8
Lymph	%	47.9	7.0
Mid	%	3.4	5.0
Gran	%	48.7	7.0

CONTROL		N	
LOT		32506-52	
		Mean ²	± Range ³
		4.14	0.18
		80.3	7.0
		12.9	4.5
		33.2	3.5
		240	30
		11.2	1.5
		8.3	0.6
		11.7	0.4
		28.3	2.0
		35.2	4.0
		4.1	0.6
		0.3	0.5
		3.9	1.0
		49.8	5.0
		3.8	5.0
		46.4	5.0

CONTROL		H	
LOT		32506-53	
		Mean ²	± Range ³
		5.09	0.22
		91.2	7.0
		12.3	4.5
		46.4	4.0
		554	60
		11.7	1.5
		20.4	1.8
		15.9	0.5
		31.2	2.0
		34.3	4.0
		13.2	1.5
		0.7	1.0
		6.5	2.5
		64.4	5.0
		3.5	5.0
		32.1	5.0

1
bg Трайност на отворен флакон 14 дни (bg)
hr Stabilität otvorene bočice 14 dni (hr)
cs Stabilita otevřené ampulky 14 dní (cs)
da Åben rør stabilitet 14 dage (da)
et Stabilsus avatud vialis 14 päevade (et)
fr Stabilité en flacon ouvert 14 jour (fr)
de Stabilität geöffneter Flaschen 14 tage (de)
el Διατήρηση δείγματος μετά την αποσφράγιση 14 ημέρα (el)
it Stabilità della fiala aperta 14 giorni (it)
lv Atvērt flakonu stabilitāti 14 dienas (lv)
lt Stabulumas atidarius buteliuką 14 dienas (lt)
no Åpen rør/glass stabilitet 14 dager (no)
pl Trwałość po otwarciu fiolek 14 dni (pl)
pt Estabilidade após abertura do frasco 14 dias (pt)
ro Termenul de valabilitate al fiolei desfăcută 14 zile (ro)
ru Стабильность вскрытого флакона 14 дней (ru)
sr Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
sk Stabilita otvorenej injekčnej liekovky 14 dní (sk)
sl Stabilita otvorenej injekčnej liekovky 14 dni (sl)
es Estabilidad de la cápsula abierta 14 días (es)
sv Hållbarhet för öppnad flaska 14 dagar (sv)
tr Açık şişe stabilitesi 14 gün (tr)

2
Средно
Sredstvo
Průměr
Middelværdi
Keskmine
Moyenne
Mittelwert
Μέση
Media
Nozīmē
Vidurkis
Middelverdi
Średnia
Média
Medie
Среднее
Srednja vrednost
Priemer
Povprečno
Media
Medelvärde
Ortalama

3
Обхват
Raspon
Rozsah
Område
Vahemik
Intervalle
Bereich
Αναμενόμενο εύρος α
Intervallo
Diapazons
Intervalas
Område
Zakres
Intervalo
Interval
диапазон
Opseg
Rozsah
Območje
Intervalo
Intervall
Aralığı

25122-4



[www.swelab.com/
support/downloads](http://www.swelab.com/support/downloads)



For Swelab Alfa / Alfa Plus with barcode reader

Alfa:

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

Alfa Plus:

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: Swelab Alfa

Lot: 32506-51

Prod.: Con-Diff Low

ID: 3250651+

Exp.: 1-Nov-2025

Instr.: Swelab Alfa

Lot: 32506-52

Prod.: Con-Diff Norm

ID: 3250652+

Exp.: 5-Nov-2025

Instr.: Swelab Alfa

Lot: 32506-53

Prod.: Con-Diff High

ID: 3250653+

Exp.: 7-Nov-2025

