

# Boule Con-Diff

CONTROL

IVD



Control	Lot	
Tri-Level	--	--
Low	22602-41	2026-07-07
Normal	22602-42	2026-07-07
High	22602-43	2026-07-07

Open vial stability 14 days <sup>1</sup>

## MEDONIC

Medonic M-series/M32		CONTROL	L
		LOT	22602-41
		Mean <sup>2</sup>	± Range <sup>3</sup>
RBC	10 <sup>12</sup> /L	2.22	0.12
MCV	fL	68.4	5.0
RDW	%	15.0	4.0
Hct	%	15.2	2.5
Plt	10 <sup>9</sup> /L	82	15
MPV	fL	11.2	1.5
WBC	10 <sup>9</sup> /L	3.3	0.3
Hgb	g/dL	5.6	0.3
MCH	pg	25.2	1.5
MCHC	g/dL	36.8	3.0
Lymph#	10 <sup>9</sup> /L	1.4	0.4
Mid#	10 <sup>9</sup> /L	0.2	0.3
Gran#	10 <sup>9</sup> /L	1.7	0.8
Lymph	%	43.0	7.0
Mid	%	6.0	5.0
Gran	%	51.0	7.0

CONTROL	N	LOT	22602-42	Mean <sup>2</sup>	± Range <sup>3</sup>
				4.19	0.18
				84.0	5.0
				13.3	4.0
				35.2	3.0
				236	30
				11.5	1.5
				8.4	0.6
				12.3	0.4
				29.4	2.5
				34.9	3.0
				3.8	0.6
				0.5	0.5
				4.1	1.0
				45.5	5.0
				6.5	5.0
				48.0	5.0

CONTROL	H	LOT	22602-43	Mean <sup>2</sup>	± Range <sup>3</sup>
				5.11	0.22
				92.3	5.0
				13.0	4.0
				47.2	3.5
				526	60
				11.7	1.5
				20.4	1.8
				16.2	0.5
				31.7	1.5
				34.3	3.0
				12.5	1.5
				1.2	1.0
				6.7	2.5
				61.3	5.0
				5.7	5.0
				33.0	5.0

	1
bg	Трайност на отворен флакон 14 дни (bg)
hr	Stabilnost otvorene bočice 14 dani (hr)
cs	Stabilita otevřené ampulky 14 dní (cs)
da	Åben rør stabilitet 14 dage (da)
et	Stabiilsus avatud vialis 14 päevade (et)
fr	Stabilité en flacon ouvert 14 jour (fr)
de	Stabilität geöffneter Flaschen 14 tage (de)
el	Διατήρηση δείγματος μετά την αποσφράγιση 14 ημέρα (el)
it	Stabilità della fiala aperta 14 giorni (it)
lv	Atvērt flakonu stabilitāti 14 dienas (lv)
lt	Stabilumas atidarius buteliuką 14 dienas (lt)
no	Åpen rør/glass stabilitet 14 dager (no)
pl	Trwałość po otwarciu fiołki 14 Liczba dni (pl)
pt	Estabilidade após abertura do frasco 14 dias (pt)
ro	Termenul de valabilitate al fiolei desfăcute 14 zile (ro)
ru	Стабильность вскрытого флакона 14 дней (ru)
sr	Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
sk	Stabilita otvorenej injekčnej liekovky 14 dní (sk)
sl	Stabilita otvorenej injekčnej liekovky 14 dni (sl)
es	Estabilidad de la cápsula abierta 14 días (es)
sv	Hållbarhet för öppnad flaska 14 dagar (sv)
tr	Açık şişe stabilitesi 14 gün (tr)

	2
	Средно
	Sredstvo
	Průměr
	Middelværdi
	Keskmine
	Moyenne
	Mittelwert
	Μέση
	Media
	Nozīmē
	Vidurkis
	Middelverdi
	Średnia
	Média
	Medie
	Среднее
	Srednja vrednost
	Priemer
	Povprečno
	Media
	Medelvärde
	Ortalama

	3
	Обхват
	Raspon
	Rozsah
	Område
	Vahemik
	Intervalle
	Bereich
	Αναμενόμενο εύρος α
	Intervallo
	Diapazons
	Intervalas
	Område
	Zakres
	Intervalo
	Interval
	диапазон
	Opseg
	Rozsah
	Območje
	Intervallo
	Intervall
	Aralığı

# For Medonic M-series / M32 with barcode reader




























**M-Series:**

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

**M32:**

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: M-series	Instr.: M-series	Instr.: M-series
Lot: 22602-41	Lot: 22602-42	Lot: 22602-43
Prod.: Con-Diff Low	Prod.: Con-Diff Norm	Prod.: Con-Diff High
ID: 2260241+	ID: 2260242+	ID: 2260243+
Exp.: 7-Jul-2026	Exp.: 7-Jul-2026	Exp.: 7-Jul-2026

1:  B o u l e C B A V 2	 B o u l e C B A V 2	 B o u l e C B A V 2
2:  2260241+	 2260242+	 2260243+
3:  26070700001153030000	 26070700002153030000	 26070700003153030000
4:  10004030000000000000	 10004030000000000000	 10004030000000000000
5:  22212684500560315225	 41918840501230435230	 51122923501620547235
6:  08215112152521536830	 23630115152941534930	 52660117153171534330
7:  03303014040020301708	 08406038060050504110	 20418125150121006725
8:  15040430700605051070	 13340455500655048050	 13040613500575033050
9:  11110082647599821517	 11110569656295138227	 11111797036702614097

# Boule Con-Diff

CONTROL

IVD



Control	Lot	
Tri-Level	--	--
Low	22602-41	2026-07-07
Normal	22602-42	2026-07-07
High	22602-43	2026-07-07

Open vial stability 14 days <sup>1</sup>

## SWELAB

Swelab Alfa/Alfa Plus	CONTROL		L
	LOT	22602-41	22602-41
	Mean <sup>2</sup>	± Range <sup>3</sup>	
RBC	10 <sup>12</sup> /L	2.25	0.12
MCV	fL	69.9	5.0
RDW	%	14.1	4.0
Hct	%	15.7	2.5
Plt	10 <sup>9</sup> /L	79	15
MPV	fL	11.8	1.5
WBC	10 <sup>9</sup> /L	3.3	0.3
Hgb	g/dL	5.7	0.3
MCH	pg	25.3	1.5
MCHC	g/dL	36.3	3.0
Lymph#	10 <sup>9</sup> /L	1.6	0.4
Mid#	10 <sup>9</sup> /L	0.1	0.3
Gran#	10 <sup>9</sup> /L	1.6	0.8
Lymph	%	47.7	7.0
Mid	%	3.9	5.0
Gran	%	48.4	7.0

CONTROL	N	
	LOT	22602-42
	Mean <sup>2</sup>	± Range <sup>3</sup>
	4.20	0.18
	85.6	5.0
	12.1	4.0
	36.0	3.0
	241	30
	12.1	1.5
	8.5	0.6
	12.4	0.4
	29.5	1.5
	34.4	3.0
	4.2	0.6
	0.3	0.5
	4.0	1.0
	49.5	5.0
	3.9	5.0
	46.6	5.0

CONTROL	H	
	LOT	22602-43
	Mean <sup>2</sup>	± Range <sup>3</sup>
	5.09	0.22
	93.8	5.0
	12.1	4.0
	47.7	3.5
	532	60
	12.3	1.5
	20.3	1.8
	16.1	0.5
	31.6	1.5
	33.8	3.0
	13.0	1.5
	0.7	1.0
	6.6	2.5
	64.2	5.0
	3.5	5.0
	32.3	5.0

1

bg	Трайност на отворен флакон 14 дни (bg)
hr	Stabilnost otvorene bočice 14 dani (hr)
cs	Stabilita otevřené ampulky 14 dní (cs)
da	Åben rør stabilitet 14 dage (da)
et	Stabiilsus avatud vialis 14 päevade (et)
fr	Stabilité en flacon ouvert 14 jour (fr)
de	Stabilität geöffneter Flaschen 14 tage (de)
el	Διατήρηση δείγματος μετά την αεροσφράγιση 14 ημέρα (el)
it	Stabilità della fiala aperta 14 giorni (it)
lv	Atvērt flakonu stabilitāti 14 dienas (lv)
lt	Stabilumas atidarius buteliuką 14 dienas (lt)
no	Åpen rør/glass stabilitet 14 dager (no)
pl	Trwałość po otwarciu fiołki 14 Liczba dni (pl)
pt	Estabilidade após abertura do frasco 14 dias (pt)
ro	Termenul de valabilitate al fiolei desfăcute 14 zile (ro)
ru	Стабильность вскрытого флакона 14 дней (ru)
sr	Proizvod stabilan nakon otvaranja pakovanja 14 dana (sr)
sk	Stabilita otvorenej injekčnej liekovky 14 dní (sk)
sl	Stabilita otvorenej injekčnej liekovky 14 dni (sl)
es	Estabilidad de la cápsula abierta 14 días (es)
sv	Hållbarhet för öppnad flaska 14 dagar (sv)
tr	Açık şişe stabilitesi 14 gün (tr)

2

	Средно
	Sredstvo
	Průměr
	Middelverdi
	Keskmine
	Moyenne
	Mittelwert
	Μέση
	Media
	Nozīmē
	Vidurkis
	Middelverdi
	Średnia
	Média
	Medie
	Среднее
	Srednja vrednost
	Priemer
	Povprečno
	Media
	Medelvärde
	Ortalama

3

	Обхват
	Raspon
	Rozsah
	Område
	Vahemik
	Intervalle
	Bereich
	Αναμενόμενο εύρος α
	Intervallo
	Diapazons
	Intervalas
	Område
	Zakres
	Intervalo
	Interval
	диапазон
	Orpsæg
	Rozsah
	Območje
	Intervalo
	Intervall
	Aralığı

## For Swelab Alfa / Alfa Plus with barcode reader

### Alfa:

1. In the Main Menu go to "Q/C Menu" "Enter Con/Cal" "Control/Calibrator Assay Value Input"
2. Scan in each blood calibrator/control column by column, sequence 1 to 9

### Alfa Plus:

1. In the Main Menu go to "Quality Control" "Input Assays"
2. Scan in each blood calibrator / control column by column, sequence 1 to 9

Instr.: Swelab Alfa  
Lot: 22602-41  
Prod.: Con-Diff Low  
ID: 2260241+  
Exp.: 7-Jul-2026

Instr.: Swelab Alfa  
Lot: 22602-42  
Prod.: Con-Diff Norm  
ID: 2260242+  
Exp.: 7-Jul-2026

Instr.: Swelab Alfa  
Lot: 22602-43  
Prod.: Con-Diff High  
ID: 2260243+  
Exp.: 7-Jul-2026

